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**PVA AND CELLULOSE PRODUCTS FOR OPHTHALMIC APPLICATIONS**  
**PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

												NO - Forsiktig: I henhold til føderale lover i USA skal dette utstyret bare selges av lege eller etter ordinasjon fra lege PL - Uwaga: Prawo federalne Stanów Zjednoczonych zezwala na sprzedaż tego urządzenia wyłącznie przez lekarza lub na zlecenie lekarza. PT – Cuidado: A Legislação Federal dos EUA restringe a venda deste dispositivo a médicos ou mediante prescrição médica. SV - Försiktighet: I USA får enligt federal lag denna produkt endast försäljas av läkare eller enligt läkares ordination.
--	--	--	--	--	--	--	--	--	--	--	--	---

GB - This Instructions for use relates to the following products:  
 CS - Tento návod k použití se vztahuje k následujícím výrobkům:  
 DE - Diese Gebrauchsanweisung bezieht sich auf die folgenden Produkte:  
 ES - Estas instrucciones de uso hacen referencia a los siguientes productos :  
 FR - Mode d'emploi des produits suivants :  
 IT - Le seguenti istruzioni per l'uso si riferiscono ai seguenti prodotti:  
 LT - Naudojimo instrukcija galioja šiems gaminiam:  
 NL - Deze gebruiksaanwijzing heeft betrekking op de volgende producten:  
 NO - Denne bruksanvisningen gjelder for disse produktene:  
 PL - Niniejsza instrukcja użytkowania dotyczy następujących produktów:  
 PT - Estas instruções referem-se aos seguintes produtos:  
 SV - Denna bruksanvisning gäller följande produkter:

GB	CS	DE	ES	FR	IT	LT	NL	NO	PL	PT	SV
PVA Spears	PVA bodce	Lanzett-Stieltupfer aus PVA	Lancetas de PVA	Lancettes PVA	Asciughini a lancetta in PVA con manico	„PVA Spears“	PVA-speertjes	PVA-spyd	Strzałki okulistyczne z PVA	Lancetas de PVA	PVA-torkar
PVA Points	PVA hroty	Lanzett-Keiltupfer aus PVA	Puntas de PVA	Pointes PVA	Asciughini a lancetta in PVA	„PVA Points“	PVA-punten	PVA-spisser	Tupfery klinowe z PVA	Pontas de PVA	PVA-spetsar
Cellulose Spears	Celulózov é bodce	Lanzett-Stieltupfer aus Cellulose	Lancetas de celulosa	Lancettes cellulose	Asciughini a lancetta in cellulosa con manico	„Cellulose Spears“	Cellulose speertjes	Cellulosespyd	Strzałki celulozowe	Lancetas de celulose	Cellulosatorkar
Cellulose Points	Celulózov é hroty	Lanzett-Keiltupfer aus Cellulose	Puntas de celulosa	Pointes cellulose	Asciughini a lancetta in cellulosa	„Cellulose Points“	Cellulose punten	Cellulosespisser	Tupfery klinowe celulozowe	Pontas de celulose	Cellulosaspetsar
PVA Corneal Light Shield	PVA rohovkový světelný štít	Hornhautlichtschutz aus PVA	Protector lumínico corneal de PVA	Lentille protectrice cornéenne PVA	Protezione corneale antiluce in PVA	„PVA Corneal Light Shield“	PVA-hoornvlieslichtscherm	Lysbeskyttelse for hornhinnen i PVA	Ostonka rogówki przed światłem z PVA	Escudo luminoso para a córnea de PVA	PVA-ljusskydd för hornhinna
PVA Eye Drain	PVA oční drén	Augendrainage aus PVA	Drenaje ocular de PVA	Drain oculaire PVA	Drenaggio oculare in PVA	„PVA Eye Drain“	PVA-oogdrain	PVA-øyedren	Dren oczny z PVA	Dreno ocular de PVA	PVA-ögonränage
PVA Eye Wick	PVA oční tampon (proužek)	Drainagedocht aus PVA	Mecha ocular de PVA	Mèche oculaire PVA	Tamponcino oculare in PVA	„PVA Eye Wick“	PVA-oogwick	PVA-øyeveke	Sączek oczny z PVA	Eye Wick de PVA	PVA-ögonveke
PVA Wick and Wipe	PVA tampon (proužek) a utěrka	Docht und Wischtuch aus PVA	Mecha y toallita húmeda de PVA	Mèche et lingette PVA	Tamponcino e salvietta in PVA	„PVA Wick and Wipe“	PVA-wick en -doekje	PVA-veke og -klut	Sączek oczny z PVA z chusteczką	Mecha e toalhete de PVA	PVA-veke och -tork
PVA Lasik Drain	PVA Lasik drén	Lasik-Drainage aus PVA	Drenaje de PVA para cirugía Lasik	Drain Lasik PVA	Drenaggio PVA LASIK	„PVA Lasik Drain“	Lasik-drain van PVA	PVA-lasik-dren	Dren Lasik z PVA	Dreno de PVA Lasik	PVA Lasik-drän
PVA Instrument Wipes	PVA nástrojov é utěrky	Instrumentenwischtücher aus PVA	Toallitas húmedas de PVA para instrumental	Lingettes PVA à instrument s	Salvietta per strumenti in PVA	„PVA Instrument Wipes“	PVA-instrumentendoekjes	PVA-instrumentkluter	Chusteczki z PVA do narzędzi	Toalhetes para instrumentos de PVA	PVA-instrumenttorkducar

**GB - INSTRUCTIONS FOR USE**
**Product Description:**

The PVA Spears, PVA Points, PVA Eye Drains, PVA LASIK Eye Drains, PVA Wicks, Cellulose Spears and Cellulose Points are sponge products used for the management of fluids during ophthalmic procedures, such as cataract and other refractive surgeries. They are supplied in various shapes and sizes according to use.

The PVA Corneal Light Shields, of various sizes, are placed on the cornea when it is necessary to moisten the cornea and shield the retina from intense operating light during ophthalmic surgery.

PVA Instrument Wipes are used to help clean medical devices after use.

**Intended Population / Patient Target Groups:**

All devices (except PVA instrument Wipes) are intended to be used on patients undergoing ophthalmic procedures. PVA Instrument Wipes are intended for use on medical devices.

**Intended Users:**

Intended for use by healthcare professionals (and also trained decontamination professionals for PVA Instrument Wipes).

**Intended Purpose:**

- Statement for Intended purpose for PVA Spears:

A body and/or body orifice contact, single-use triangular piece of medical grade, highly absorbent PVA supplied with attachment to a handle. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. Can also be used for tissue manipulation. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. There are no known contraindications for these devices.

- Statement for Intended purpose for PVA Points:

**'PVA AND CELLULOSE PRODUCTS FOR OPHTHALMIC APPLICATIONS  
'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

A body and/or body orifice contact, single-use triangular piece of medical grade, highly absorbent PVA. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. Can also be used for tissue manipulation. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. There are no known contraindications for these devices.

- **Statement for Intended purpose for Cellulose Spears:**

A body and/or body orifice contact, single-use triangular piece of medical grade, highly absorbent cellulose supplied with attachment to a handle. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. Can also be used for tissue manipulation. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. They are contraindicated for patients undergoing LASIK surgery.

- **Statement for Intended purpose for Cellulose Points:**

A body and/or body orifice contact, single-use triangular piece of medical grade, highly absorbent cellulose. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. Can also be used for tissue manipulation. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. They are contraindicated for patients undergoing LASIK surgery.

- **Statement for Intended purpose for PVA Eye Drains:**

A body and/or body orifice contact, single-use rectangular strip of medical grade, highly absorbent PVA which drains into an attached polythene bag which has a self-adhesive strip to permit it to be secured to the drape during ophthalmic surgical procedures. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. The PVA portion of the drain has a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. There are no known contraindications for these devices.

- **Statement for Intended purpose for PVA LASIK Eye Drains:**

A body and/or body orifice contact, single-use piece of medical grade, highly absorbent PVA used during ophthalmic surgical procedures. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. They can be used to separate the flap from the cornea along the hinge during LASIK procedures. The PVA LASIK eye drain has a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. There are no known contraindications for these devices.

- **Statement for Intended purpose for PVA Wicks:**

A body and/or body orifice contact, single-use rectangular strip of medical grade, highly absorbent PVA used during ophthalmic surgical procedures. Intended to be used in a sterile condition by healthcare professionals on any patient for the management of fluids, and to staunch blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye), cornea and/or surrounding tissue. There are no known contraindications for these devices.

- **Statement for Intended purpose for PVA Instrument Wipes:**

A non-contact, single-use sheet of PVA intended to be used in a sterile condition by healthcare professionals to help clean medical devices prior to full decontamination. The instrument wipe is only to be handled with medical grade gloves and as such is non-contact with a cumulative transient use of less than 60 minutes. There are no known contraindications for this device. These devices are not intended to be used on a patient.

- **Statement for Intended purpose for PVA Corneal Light Shields:**

A body orifice contact, single-use circular shaped piece of medical grade and highly absorbent PVA. It is intended to be used by healthcare professionals in a sterile condition on any patient for keeping the surface of the cornea moist and cool during ophthalmic procedures, and for shielding the retina from intense operating light. They have a cumulative transient use of less than 60 minutes on the conjunctiva (mucous membrane of the eye) and/or the cornea. There are no known contraindications for these devices.

**Indications for use:**

- **PVA Spears, PVA Points, Cellulose Spears, Cellulose Points:**

Indicated for all microsurgical ophthalmic procedures for tissue manipulation and management of fluids in the operative field.

The use of these products is based on their ability to hydrate quickly and efficiently. Used to absorb any blood seeping from the operative site or excess saline used during the procedure. The sponge comes in contact with the tissue but is not invasive of the wound. The design of the sponge enables the surgeon to precisely place the very tip onto the operative site and for hydration to take place quickly from that point. No medications or other substances should be delivered by these products.

The Cellulose medical devices are not recommended for LASIK procedures.

- **PVA Drains, PVA LASIK Drains and PVA wicks:**

Indicated for all microsurgical ophthalmic procedures for tissue manipulation and management of fluids in the operative field. The wick is placed between the eyeball and the lower eyelid to drain away excess liquid from the operative site or injury. Saline can be used to moisten the wick prior to placement to increase the speed of the wicking.

- **PVA Corneal Light Shields:**

Indicated for placement on the cornea to moisten the cornea and shield the retina from intense operating light during ophthalmic surgery.

Usually placed on the cornea during a break in the operating procedure to protect the delicate internal membranes of the eye from damage associated with the heat generated from the microscope light beam. Also maintains moisture on the surface of the cornea.

- **PVA Wicks and Wipes:**

A combination product of an instrument wipe and an eye wick. The wick can be detached and placed between the eyeball and the lower eyelid to drain away excess liquid from the operative site or injury. Saline can be used to moisten the wick prior to placement to increase the speed of the wicking. Instrument wipes are used to help clean medical devices.

- **PVA Instrument Wipes:**

Instrument wipes are intended to be used for the cleaning of medical devices.

**Contraindications:**

Cellulose products are contraindicated on patients undergoing LASIK procedures. PVA Instrument Wipes are not intended for use on patients.

**Side effects:**

Possible side effects, for PVA Spears, PVA Points, PVA Eye Drains, PVA LASIK Eye Drains, PVA Wicks, PVA Corneal Light Shields, Cellulose Spears and Cellulose Points (products whose use contact the eye), include:

- Scratching of the ocular mucous membrane by direct excessive forces from a dry device being applied to the eye
- Irritation if any loose particles are not rinsed away from the treatment site
- Cross infection / contamination from incorrect handling
- Foreign body granuloma if any loose particles are not rinsed away from the treatment site
- Bleb inflammation if any loose particles contaminate in or around a bleb

**Clinical benefits:**

PVA Instrument Wipes are not used in a clinical application.

- **All patient contact PVA products**
  - Manages fluids and staunches blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery.
  - Can be used for tissue manipulation.
  - Ultra-fast wicking action.
  - Highly absorbent.
  - Supplied compressed but expands immediately upon contact with body fluids, saline or balanced salt solution, wicking 10-15 times its own weight away from the operative site.
  - Supplied sterile, single use only, declared 5-year shelf life.
- **Patient contact PVA Eye Drains:**
  - Available with either 80cc or 400cc capacity bag which has a self-adhesive strip to permit it to be secured to a drape.
  - Supplied sterile, single use only, declared 5-year shelf life
- **Patient contact PVA LASIK Eye Drains**
  - Can be used to separate the flap from the cornea along the hinge.
  - Supplied sterile, single use only, declared 5-year shelf life
- **Patient contact PVA Corneal Light Shields:**
  - Supplied sterile, single use only, declared 5-year shelf life.
  - Used to keep the cornea moist and cool during ophthalmic procedures.
  - Shields retina from intense operating light.
- **Cellulose Spears, Cellulose Points:**
  - Manages fluids and staunches blood loss after invasive surgery or traumatic injury in the areas of ophthalmic surgery.
  - Can be used for tissue manipulation.

**'PVA AND CELLULOSE PRODUCTS FOR OPHTHALMIC APPLICATIONS**  
**'PRODUCT INFORMATION AND INSTRUCTIONS FOR USE**

- Constructed from medical grade biocompatible cellulose.
- Highly absorbent.
- Supplied sterile, single use only, declared 5-year shelf life.

**CAUTIONS:**

- These are delicate medical devices and should be handled with care..
- Sterility and performance are not guaranteed on devices that have passed their Use By date or if the packaging or seals have been damaged. If label including use by date is illegible, do not use.
- As with all PVA and cellulose products loose particles may occur, these should be rinsed from the treatment site.
- Do not cut the device.
- The product comes into contact with bodily fluids, which can be contaminated. Care should be taken in the handling and disposal of the device after use to prevent contamination. These devices should be disposed of after use by following your facilities approved procedures for contaminated waste.
- Control use of devices to ensure none are lost during the procedures, devices do not show up on X-Ray or MRI scanners. Check quantities are correct.

**Incident Reporting:**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the country in which the user and/or patient is established.

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